



# HOW TO CONFIGURE

EYE GAZE ON THE **AUTONOME**

# STEP BY STEP

This document will give autoME users & their caregivers step by step instructions on how to configure their eye gaze device by accessing the settings menu & calibrating their positioning.

## GET STARTED

---

- 1 To configure eye gaze, go to "Settings"
- 2 In the "Settings" panel, select "Pointer Settings"
- 3 Then select "Enable Eye Gaze"
- 4 Once Eye Gaze is enabled, select "Calibrate Eye Gaze"
- 5 Follow the prompts provided by the device
- 6 After completing Eye Gaze calibration select "Go Back", then "Go Back" again
- 7 Then select "Minimize Window" & the Eye Tracking Settings should be visible
- 8 To go back to the autoME software select "Main Menu"



ACCESSIBILITY  
SERVICES, INC.

# EYE TRACKING SETTINGS APP


If Eye Gaze has already been enabled & you would like to view the Eye Tracking Settings App follow these steps:


## GET STARTED

---

- 1 Go to "Settings" then "Advanced Settings"
- 2 Ensure the "Desktop Items" box is checked under "Application Manifest"
- 3 Go back to the "Main Page"
- 4 Select "Desktop Items"
- 5 Select "Eye Tracking Settings"
- 6 The settings page will open & the "Eye Tracking Settings" will be visible
- 7 To go back, select "Main Menu" at the top right

## GET IN TOUCH

 @accessibilityservices

 @accessibilityservices

CONTACT US AT 800 933 8400  
OR VISIT [ACCESSIBILITYSERVICES.COM](https://www.accessibilityservices.com)

# REFERENCE IMAGES



1




Settings Menu

4



Tobii Eye Tracking Page

2

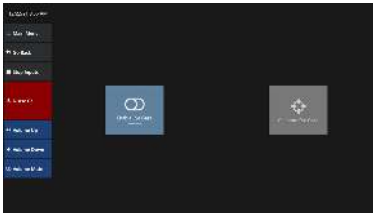


Pointer Settings

5



autonOME Main Menu

3

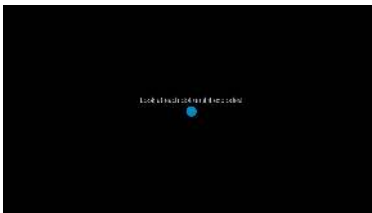


Calibration Page

6



autonOME Advanced Settings Page

7

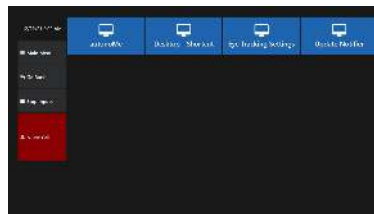

Desktop Items Menu